h/p/cosmos robowalk® 强身器 手册

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h/p/cosmos robowalk® 强身器

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h/p/cosmos robowalk® 强身器 f

h/p/cosmos robowalk® 强身器 b

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V1

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[cos30022man-cn]

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公司名称: h/p/cosmos sports & medical gmbh

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Management Service

CERTIFICATE

The Certification Body
of TÜV SÜD Management Service GmbH
certifies that

h/p/cosmos sports & medical gmbh
Am Sportplatz 8
D-83365 Nußdorf

has established and applies
a Quality Management System for

**Design and development, production, distribution and service of
treadmill ergometers, ergometers, diagnosis,
therapy and training systems**

An audit was performed, Report No. **70078857**
Proof has been furnished that the requirements
according to

ISO 9001:2008

are fulfilled. The certificate is valid until **2013-11-30**

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CERTIFICATE

No. Q1N 10 12 45283 007

Holder of Certificate: h/p/cosmos
sports & medical gmbh

Am Sportplatz 8
83365 Nussdorf - Traunstein
GERMANY

Facility(ies): h/p/cosmos sports & medical gmbh
Am Sportplatz 8, 83365 Nussdorf - Traunstein, GERMANY

Certification Mark:

Scope of Certificate: Design and development, production,
distribution and service of treadmill
ergometers, ergometers, diagnosis
and therapy systems

Applied Standard(s): EN ISO 13485:2003/AC:2009
Medizinprodukte – Qualitätsmanagementsysteme –
Anforderungen für regulatorische Zwecke
Medical Devices – Quality Management Systems –
Requirements for regulatory purposes

The Certification Body of TÜV SÜD Product Service GmbH certifies that the company mentioned above has established and is maintaining a quality system which meets the requirements of the listed standard(s). See also notes overleaf.

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President & CEO / R & D Manager

Wolfgang Petry

Safety Representative (Medical Devices § 30)

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h/p/cosmos service report form no.:

☐ Repair	☐ Service	☐ Maintenance contract	☐ Warranty enquiry	☐ Good will bases
Company / organisation			Customer number	
Contact Person			Location, department, room #	
Street			Business hours	
ZIP code / city				
Phone / ext.			Device type	
Fax			Serial number (important!)	
email			Date of delivery	
www.			Distributor company name	

Detailed description of error and who detected the error. Without serial number of the device no further processing is possible.

--

Data of running machine or device, read out of MCU / User Terminal

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Following minimum actions have to be performed and checked. Confirm all performed actions with: X

Running machine cleansed inside / outside	Motor / fan grid cleansed	Light barrier cleaned
Driving belt tension / position adjusted	Light barrier fixation and adjustment	Speed + elevation sensor adjustment / fixation
Running belt oiled with ml	Running belt tension adjusted rotations	Running belt position adjusted: turns to
Elevation device tested	Elevation device greased (spindle + cogwheel)	All fixation washers of the roller axle checked
Function test mechanical and electrical	Function MCU / UserTerminal / Display	Test running (with person kg) min.
Pulse measurement ☐ yes ☐ no	Pulse control ☐ yes ☐ no	Function test pulse measurement
OP 47 intervall display reset	Error memory frequency inverter ☐ erased	Maximum speed: km/h

Notes:

Firmware update

The Firmware of the product ☐ updated to latest version _____ ☐ the latest version is already installed ☐ update rejected by customer

All spare parts needed: including order number as well as serial number for replaced and new parts

Amount	Spare part including date of production	Order number	Serial number replaced	Serial number new	Remark

Additional costs ☐ none ☐ lubrication and cleaning material ☐ delivery costs spare parts ☐ fastening material ☐ stickers as following list:

cos10144 address sticker	cos10253-01 operation / safety instr.	cos10508 caution danger zone
cos10512 running belt adjustment	cos11668 safety warnings LT	cos11787 base sticker annual check
cos11880 unplug before opening	cos12976 warning triangle sticker	cos14543 top sticker year

Control check-up according to VDE 0701 / VDE 0751

☐ The checks, necessary after every service (even mechanical service!) have been performed according to VDE 0701 (sports equipment), respectively VDE 0751 (medical device) after finishing the service with separate protocol form cos15533 & the following reported measurements:

Final report

☐ The problem described above could be solved and the running machine / device can be used without limitations. Date for next check _____

☐ The problem described above is not solved completely. Device can be used but following actions are required: _____

☐ The device is defective and not safe! The device has been marked, put out of operation, secured against operation and must not be used at all!

Start working: o'clock	Working time: hours	☐ Distance flat rate for km
Driving distance (one way): km	Driving time (one way): hours	☐ Total distance ☐ Distance proportional

Place, date	Stamp / Signature customer, name in capital letters	Signature technician, name in capital letters

Original to h/p/cosmos factory in Germany Pink print copy for technician Blue print copy for customer
h/p/cosmos sports & medical gmbh Am Sportplatz 8 DE 83365 Nussdorf-Traunstein phone +49 / 86 69 / 86 42 0 fax +49 / 86 69 / 86 42 49 service@h-p-cosmos.com www.h-p-cosmos.com

revision Feb. 09, 2006
[cos15531] service report form

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